

- Prevents the occurrence of bed sores due to continuous lying
- Prevents the occurence of infections
- Improves the health quality of patient
- In treatment it provides pharmacoeconomic
- With children and adult sizes it is suitable for all patient groups
- It is suitable for the patients up to 220 kg
- Easy and pratic usage



Usage:

Usage: Spread a thin linen on sick bed. (May be patients mattress) air inlet hoses are being connected to the pump and starts to work at maximum air pressure for nearly around 20 minutes. Within this time air cells are full-filled and air pressure is being adjusted manually over motor pump depending on the kilo of the patient. The motor goes on operating the system with an automatically change row. For an emergency case there is a CPR emergency air drainage tap. Switching-on the CPR button air drains automatically.

1- Doç. Dr. Oya Oğuz. İ.Ü. Cerrahpaşa Tıp Fakültesi Sürekli Tıp Eğitimi Etkinlikleri. Geriatrik Hasta ve Sorunları Sempozyumu.12-13 Kasım 1998, İstanbul, s. 147-151



Üretici Firma - Vizyon İç ve Dış Ticaret A.Ş.

Ticaret Cd. No: 143-145 Giyimkent - Esenler / İstanbul

Tel: (+90) 212 438 24 70 (Pbx) Fax: (+90) 212 438 24 73



"FOR THE PREVENTION OF DECUBITUS ULCERS"





Decubitus ulcers:

Deep ulcers which occurs because of high risc factors like senility, norolojic defects malnutrition and immobility.



In Decubitus ulcer **bacterial** infection seen often¹

To avoid Decubitus ulcer the **bed** being used is very important



With anti decubitus feature avoids bed sores

With ventilation feature in bed provides skin breathe and prevents perspiration

Thanks to the oxygen system in bed, accelerates recovery of different sores

Thanks to the compressed air pump, when A tubes are full five minutes along; B tubes are empty. At the end of fifth minute empty B tubes fills automatically and A tubes gets drain off. Thus, different points of body is not under the same pressure. With this system the blood pressure is being organised well and it prevents the occur of decubitus ulcer.